

# Needle-Pro® EDGE™

## Manual Safety Needle and Syringe Devices



### Why choose safety needle and syringe devices from Cardinal Health?

Our line of manual safety syringes provides you with a single source to meet all your safety technology needs. In addition to being a safer alternative to traditional syringes, our syringes offer excellent performance characteristics, such as low waste space and easy-to-read graduations.

#### Performance features

##### Safety technology

- Easy-to-use
- Single-handed activation system
- Audible safety feature provides confirmation of sheath activation
- Sleek, low-profile design offers unobstructed view of needle

Constructed with user safety in mind, The Needle Pro® EDGE™ allows your hand to remain behind the needle at all times – your finger never has to move towards the needle. The sleek, low-profile design of the EDGE™ safety device provides clinicians with an unobstructed view of the needle. Clear line of sight helps clinicians administer injections efficiently, as well as draw medications easily.

##### Reduction in medication costs

- “True needle length” with EDGE™ safety technology eliminates the extra dead space introduced by some other designs
- Fully integrated needle and safety device
- More efficient utilization of medications means greater cost savings for your institution

Other safety devices may add dead space to the needle which means that with every injection you waste valuable medication. When you're giving thousands of injections a year, dead space adds up to a major cost impact. The design of the EDGE™ safety devices combines the advantages of safety with the efficiency of a conventional needle.

##### Graduations and color coding

- Optically clear precision-marked barrels
- Complete color coding of safety devices
- Clear, distinctive color coding on packaging

Clinicians need color coding that's highly visible, even at a distance. Using a proprietary manufacturing process, EDGE™ safety devices are completely embedded with color, which is designed to stand out.



#### How to use the safety device



**Step 1**  
Perform injection as per normal procedure.



**Step 2**  
Upon completion of procedure, engage the device using a one-handed technique by gently pressing the sheath against a flat surface. DO NOT use free hand to press the sheath over the needle.



**Step 3**  
Immediately dispose of device into an approved sharps container.

# Ordering information

All safety needle and syringe devices are not made with natural rubber latex.

## EDGE™ Safety Devices

Cat. No.	Size	Color	Qty.
<b>Devices only</b>			
ED181-NO	18G x 1 in.	Pink	100 ea/bx, 1,000 ea/cs
ED1815-NO	18G x 1½ in.	Pink	100 ea/bx, 1,000 ea/cs
ED191-NO	19G x 1 in.	Cream	100 ea/bx, 1,000 ea/cs
ED201-NO	20G x 1 in.	Yellow	100 ea/bx, 1,000 ea/cs
ED2015-NO	20G x 1½ in.	Yellow	100 ea/bx, 1,000 ea/cs
ED211-NO	21G x 1 in.	Green	100 ea/bx, 1,000 ea/cs
ED2115-NO	21G x 1½ in.	Green	100 ea/bx, 1,000 ea/cs
ED221-NO	22G x 1 in.	Black	100 ea/bx, 1,000 ea/cs
ED2215-NO	22G x 1½ in.	Black	100 ea/bx, 1,000 ea/cs
ED231-NO	23G x 1 in.	Blue	100 ea/bx, 1,000 ea/cs
ED2315-NO	23G x 1½ in.	Blue	100 ea/bx, 1,000 ea/cs
ED2558-NO	25G x ⅝ in.	Orange	100 ea/bx, 1,000 ea/cs
ED251-NO	25G x 1 in.	Orange	100 ea/bx, 1,000 ea/cs
ED2515-NO	25G x 1½ in.	Orange	100 ea/bx, 1,000 ea/cs
ED2705-NO	27G x ½ in.	Gray	100 ea/bx, 1,000 ea/cs
ED27125-NO	27G x ¼ in.	Gray	100 ea/bx, 1,000 ea/cs

## Devices with 3mL Luer Lock Syringe

Cat. No.	Size	Color	Qty.
ED3201-CM	20G x 1 in.	Yellow	50 ea/bx, 400 ea/cs
ED3211-CM	21G x 1 in.	Green	50 ea/bx, 400 ea/cs
ED32115-CM	21G x 1½ in.	Green	50 ea/bx, 400 ea/cs
ED32215-CM	22G x 1½ in.	Black	50 ea/bx, 400 ea/cs
ED3231-CM	23G x 1 in.	Blue	50 ea/bx, 400 ea/cs
ED32558-CM	25G x ⅝ in.	Orange	50 ea/bx, 400 ea/cs
ED3251-CM	25G x 1 in.	Orange	50 ea/bx, 400 ea/cs

## Needle-Pro® Fixed Needle Syringes

Cat. No.	Size	Color	Qty.
ED12558-TB	1mL, 25G x ⅝ in. regular bevel needle	Orange	100 ea/bx, 600 ea/cs
ED12638-TB	1mL, 26G x ⅝ in. intra-dermal bevel needle	Brown	100 ea/bx, 600 ea/cs
ED12705-TB	1mL, 27G x ½ in. regular bevel needle	Gray	100 ea/bx, 600 ea/cs
<b>U-100 Insulin Syringes</b>			
ED052805IN	0.5mL, 28G x ½ in.	Insulin orange	100 ea/bx, 600 ea/cs
ED12805-IN	1mL, 28G x ½ in.	Insulin orange	100 ea/bx, 600 ea/cs
ED032905IN	0.3mL, 29G x ½ in.	Insulin orange	100 ea/bx, 600 ea/cs
ED052905IN	0.5mL, 29G x ½ in.	Insulin orange	100 ea/bx, 600 ea/cs
ED12905-IN	1mL, 29G x ½ in.	Insulin orange	100 ea/bx, 600 ea/cs
ED033005IN	0.3mL, 30G x ½ in.	Insulin orange	100 ea/bx, 600 ea/cs
ED053005IN	0.5mL, 30G x ½ in.	Insulin orange	100 ea/bx, 600 ea/cs

## Syringes only

Cat. No.	Size	Qty.
<b>Luer Lock</b>		
SY35003LL	3mL	100 ea/bx, 3,000 ea/cs
SY35005LL	5mL	100 ea/bx, 2,000 ea/cs
SY35010LL	10mL	100 ea/bx, 1,200 ea/cs
SY35020LL	20mL	100 ea/bx, 900 ea/cs
SY35030LL	30mL	50 ea/bx, 450 ea/cs
SY35060LL	60mL	50 ea/bx, 400 ea/cs
<b>Luer Slip</b>		
SY35001LS	1mL	100 ea/bx, 4,000 ea/cs
SY35003LS	3mL	100 ea/bx, 3,000 ea/cs
SY35005LS	5mL	100 ea/bx, 2,000 ea/cs
SY35010LS	10mL	100 ea/bx, 1,200 ea/cs
SY35020LS	20mL	100 ea/bx, 900 ea/cs
SY35030LS	30mL	50 ea/bx, 450 ea/cs
SY35060LS	60mL	50 ea/bx, 400 ea/cs

For more information or to order, contact your Cardinal Health sales representative, call 877.CARDINAL or visit [cardinalhealth.com](http://cardinalhealth.com).

EDGE and Needle-Pro are trademarks of the Smiths Medical family of companies.

© 2014 Cardinal Health. All rights reserved. CARDINAL HEALTH, the Cardinal Health LOGO, ESSENTIAL TO CARE are trademarks or registered trademarks of Cardinal Health. All other marks are the property of their respective owners. Lit. No. 2MC14-22607 (10/2014)



[cardinalhealth.com](http://cardinalhealth.com)