



Cardinal Health™ Monoject™ Magellan™ Hypodermic Safety Needle

Instructions for use



1 Select the appropriate Monoject™ Magellan™ Hypodermic Safety Needle for your procedure. All Monoject™ Magellan™ Safety Needles are ISO color-coded.



2 Open package using an aseptic technique by peeling back paper tab and remove safety needle.



3 Secure the needle by holding the needle and twisting the syringe clockwise to fully seat the needle. Remove protective needle sheath.



4 Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.

Lock the safety shield using any of the following activation methods:



5 Thumb



Finger



Flat surface



6 Verify locked position through audible and tactile "click." Locked position will completely cover needle tip.



7 Once safety shield is locked, immediately dispose of needle and syringe in an approved sharps container.

For more information, visit cardinalhealth.com/monoject
To place an order, log onto The Cardinal Health MarketSM or
call Customer Service at **1.800.964.5227**

