



# Cardinal Health™ Monoject™ Magellan™ Insulin and Tuberculin Safety Syringe

## Instructions for use



1 Select the appropriate Monoject™ Magellan™ Insulin or Tuberculin Safety Syringe for your procedure.



2 Open package by peeling back paper tabs and remove syringe.



3 Remove protective needle sheath.



4 Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.



5 Lock the safety shield using your thumb.

or



Your finger.



6 Verify locked position through audible and tactile "click." Locked position will completely encase needle.



7 Once the shield is locked, immediately dispose of syringe in an approved sharps disposal container.

For more information, visit [cardinalhealth.com/monoject](http://cardinalhealth.com/monoject)  
To place an order, log onto The Cardinal Health Market<sup>SM</sup> or  
call Customer Service at **1.800.964.5227**

