



Cardinal Health™ Monoject™ Hypodermic Safety Needle

Instructions for use



Select the appropriate Monoject™ Hypodermic Safety Needle for your procedure. All Monoject™ Safety Needles are ISO color-coded.



Open package using aseptic technique by peeling apart the plastic and paper tab and remove safety needle.



Secure the needle by holding the needle and twisting the syringe clockwise to fully seat the needle. Remove protective needle sheath.



Draw up medication and administer injection using aseptic technique in accordance with your facility's protocol.

Instructions for disposal



Lock the safety shield using either of the activation methods.



An audible and tactile click will verify the safety device has been activated.



Once safety shield is locked, immediately dispose of needle and syringe in an approved sharps container.

For more information, visit cardinalhealth.com/monoject
To place an order, log onto The Cardinal Health MarketSM or
call Customer Service at **1.800.964.5227**