



# Cardinal Health™ Monoject™ Insulin and Tuberculin Safety Syringe

## Instructions for use



1 Remove syringe from package and discard tip cap. Recapping is not necessary with this syringe.




2 Draw medication from vial according to your facility's protocol.




3 To transport syringe, extend the shield forward until you hear and feel a click. The needle is now protected for transport but the syringe is not permanently locked.



4 To give injection, slide shield back to its original position exposing the needle. Perform injection following facility protocol.

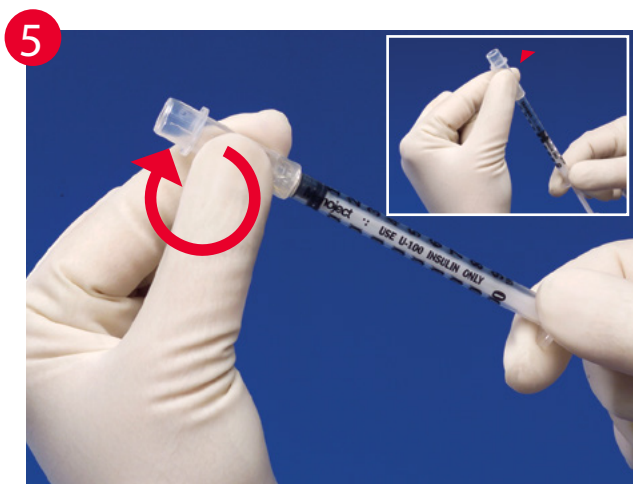

5 To permanently lock syringe, extend shield forward until it clicks, then twist the shield in either direction until it clicks again to indicate final-lock.

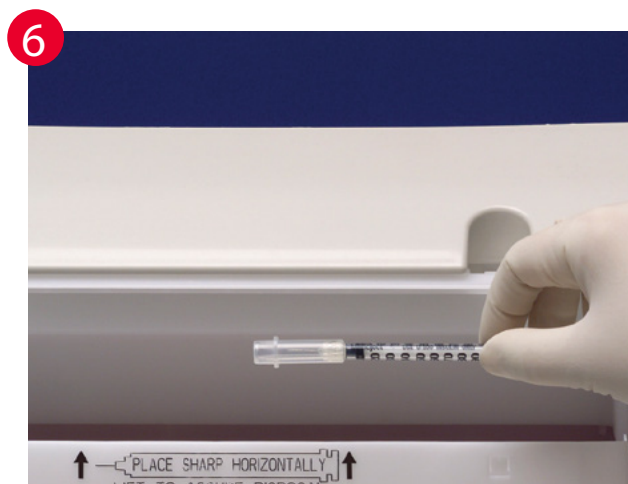

6 Discard permanently locked syringe in an approved sharps container.

For more information, visit [cardinalhealth.com/monoject](https://www.cardinalhealth.com/monoject)  
To place an order, log onto The Cardinal Health Market<sup>SM</sup> or  
call Customer Service at **1.800.964.5227**