

# Cardinal Health™ Monoject™ SmarTip™ Needleless Vial Access Cannula

## Instructions for use



1 Remove Monoject™ SmarTip™ cannula from packaging. Use aseptic technique.



2 Clean medication vial per facility protocol.




3 Penetrate vial with Monoject™ SmarTip™ cannula. For best results place vial on counter. DO NOT angle cannula during insertion; hold perpendicular to vial. Use quick, consistent force to penetrate center of vial.



4 Draw medication per facility protocol. If pressurizing vial, use air equal to amount of medication to be drawn to avoid over-pressurizing vial and expulsion of excess fluid.



5 Re-sheath Monoject™ SmarTip™ cannula using a one-handed OSHA approved scoop and hold technique.

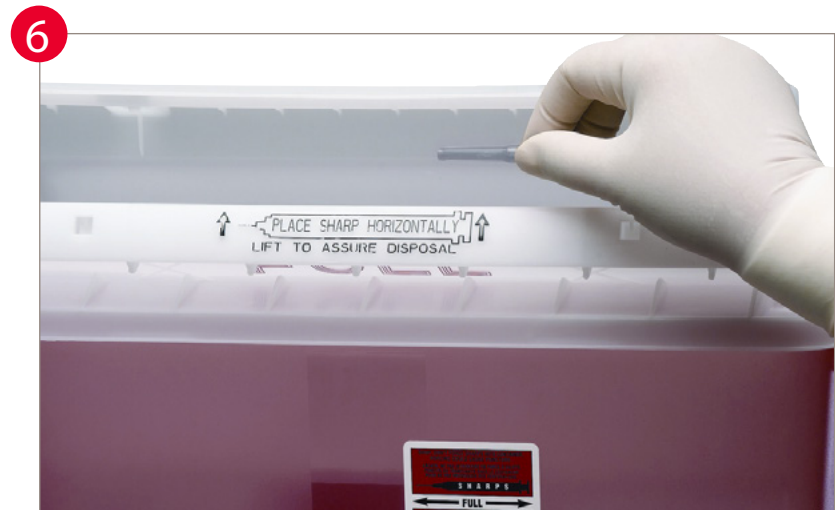

6 Remove cannula and dispose in approved sharps container prior to medication delivery.

For more information, visit [cardinalhealth.com/monoject](http://cardinalhealth.com/monoject)  
To place an order, log onto The Cardinal Health Market<sup>SM</sup> or  
call Customer Service at **1.800.964.5227**

